

Cleaning Guidelines for Jaco EHR Workstations



Simple. Solid. Smart.

All metal surfaces of your Jaco workstation are treated with a smooth, high-gloss, non-porous antimicrobial silver ion powder coat that either kills or inhibits the spread of infection-causing microbes. For maximum infection control and continued product performance, Jaco recommends you routinely clean and disinfect your workstations in accordance with your hospital's infection control program.

Clean by Hand

Jaco's smooth, high-gloss antimicrobial finish is very easy to clean by hand. Follow these guidelines for the best results:

1. Make sure the product is disconnected from the AC power outlet before cleaning.
2. Use an approved disinfectant wipe, or a soft clean cloth and approved disinfectant (see below).
3. Wipe down all exposed surfaces of the workstation as needed.
4. Make sure the workstation is fully dry before plugging it back into the power source.

Approved chemicals, detergents and disinfectants for hand-cleaning Jaco products

- 10% or less bleach solution
- Isopropyl alcohol
- Metrex Cavi Wipes
- Clorox hydrogen peroxide disinfectants
- Clorox germicidal wipes (active ingredient: sodium hypochlorite (bleach))
- Mild, non-abrasive detergent and water

UV Sterilization

Jaco's powder coated aluminum and steel body construction is exceptionally well-suited to UV sterilization. Specifically, Jaco's antimicrobial powder coat will tolerate up to 250 hours of constant UV-B exposure, or up to 24 hours of constant UV-C exposure, with minimal impact on gloss, color and film integrity. Most UV-C sterilization systems require 90 seconds to 30 minutes of exposure.



COVID-19: Check with your disinfectant or sterilization supplier

- Many detergent and disinfectant labels claim effectiveness against coronavirus, but many of these labels were printed prior to the advent of the COVID-19 strain. **Contact the manufacturer to verify their product has been tested and found effective against COVID-19.**
- Many manufacturers of UV sterilization equipment are promoting their products' effectiveness against COVID-19. **Contact the manufacturer to verify effectiveness and required exposure times.**

QUESTIONS? Contact your Jaco representative, or call Jaco directly at 800-649-2278.